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CELIMED® U-615
(model NB-152U)

CZ

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EN Instruction manual

IMPORTANT INSTRUCTIONS:


- 1) Always insert only 1 piece of medication cup into the nebulizer. The device comes with 5 pcs inserted into each other, therefore it is important to only take out 1 piece when starting to use the device or when exchanging old / damaged medication cup for a new one.
- 2) When separating the medication cups from each other, please do not use much force, otherwise You could damage the medication cup. Damaged medication cups do not work at all or cause a production of limited amount of aerosol.
- 3) Do not use much force to attach the nebulizing compartment cover- please see chapter 5.2. During the attachment the transparent arrow has to sit against the blue symbol of the opened lock. Then slightly turn the cover to the right side to align with the blue symbol of closed lock. In case You do not attach the cover properly or use a big force, the bottom part with little plastic protrusions can get damaged!
- 4) In case You are using Vincentka, it is important for the liquid not to be sparkling. When using a fresh bottle of Vincentka, shake it carefully for the liquid to become flat.
- 5) In case You store the Vincentka or other medication refrigerated, please make sure to bring it to the room temperature before use.

CONTENT

EN


1. PARTS AND COMPONENTS	2
2. SYMBOLS	3
3. GENERAL	3
4. IMPORTANT SAFEGUARDS	4
4.1 PREPARATION	4
4.2 OPERATION	4
4.3 CLEANING	4
5. OPERATION PROCEDURE	5
5.1. FILL WATER RESERVOIR	5
5.2. FILL MEDICATION CUP	5
5.3. CONNECT EXTERNAL POWER	6
5.4. TURN ON THE POWER AND ADJUST NEBULIZING RATE	6
5.5. ADJUST THE AIR FLOW RATE	6
5.6. ASSEMBLE THE MASK (OR MOUTHPIECE) AND BEGIN TREATMENT	7
6. DISPOSE EXCESS MEDICATION AND WATER	7
7. CLEANING	7
8. SPECIFICATIONS	8
9. TROUBLESHOOTING	8

1. PARTS AND COMPONENTS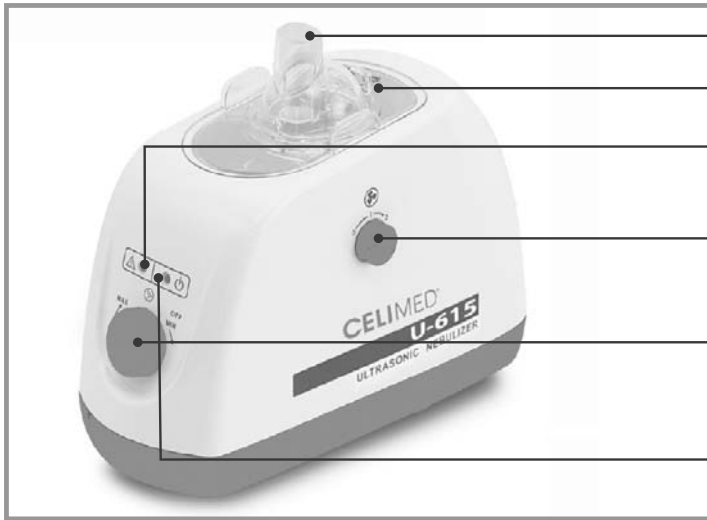


- Nebulizing Compartment
- Water reservoir Cover
- Water Level Alarm Light
- Air Flow Adjusting Knob
- ON/OFF and Nebulizing Rate Adjusting Knob
- Power Indicator

Main Body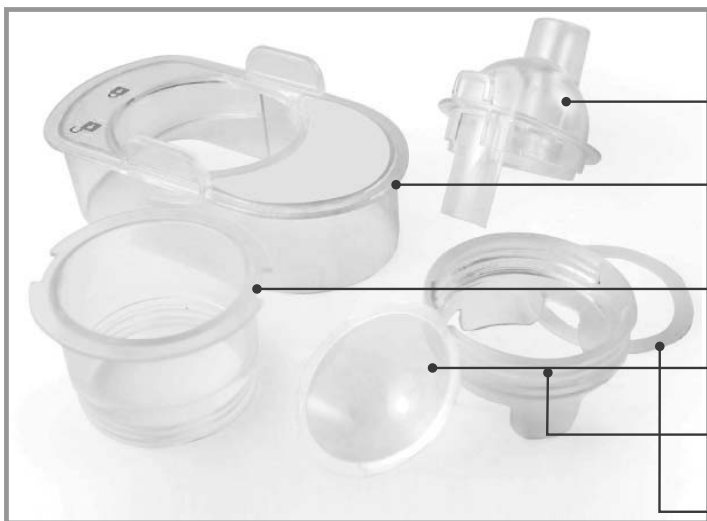

- Cover of Nebulizing Compartment
- Water Reservoir Cover
- Middle Shell of Nebulizing Compartment
- Medicine Cup
- Bottom Shell of Nebulizing Compartment
- Seal Ring

Nebulizing Compartment
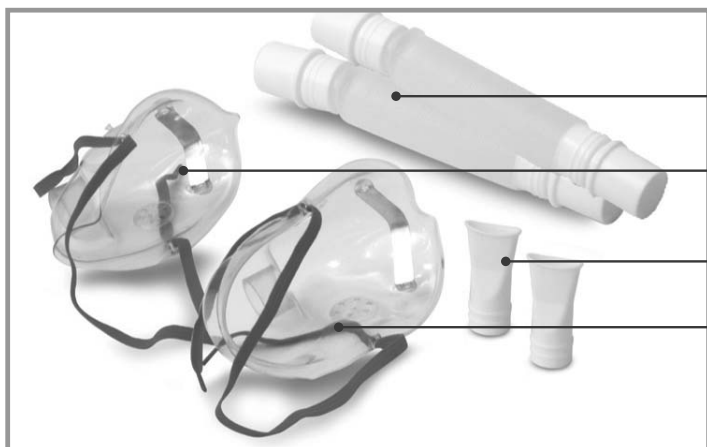
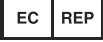
- Inhalation hose 2x
- Child mask
- Mouthpiece 2x
- Adult mask

Included accessories

EN

2. SYMBOLS

Symbols	Meaning
	Manufacturer
	EU-representative
	DISPOSAL PROCEDURE (Directive 2012/19 / EU-WEEE) Symbol for the marking of electrical and electronics devices according to Directive 2002/96/EC. The device, accessories and the packaging have to be disposed of waste correctly at the end of the usage. Please follow Local Ordinances or Regulations for disposal.
	CLASS II
	CE marking in conformity with EC directive 93/42/EEC
	Medical device
Classification IP	IP 21
	Attention, consult accompanying documents
	Type BF Applied Part

EN

3. GENERAL

ULTRASONIC NEBULIZER CELIMED U-615 (NB-152U)

Thank you for purchasing this product. To ensure the correct use of this product, please read carefully the following instructions, non-compliance could result in damages to persons or objects. Before using the appliance, please check that the unit is in perfect conditions, without any visible damages. Otherwise contact the authorized service - The contact can be found in the end of this instruction manual.

THERAPEUTIC USE

This is a medical device and therefore it must be used based on given instructions from Your doctor or pulmonary specialist. It must be used according to the instructions, contained in this manual and it is important that the user reads and understands the use and maintenance information of the device. This device has been manufactured exclusively for domestic aerosol therapy treatment. It must be used for nebulizing medicines such as antibiotics and bronchodilators. All other purposes must be considered improper.

WARRANTY

The warranty for the main unit is two years. The warranty does not cover the consumer material (accessories and consumer parts). Warranty will not be accepted if tempered with the device by using it in any other way than described by the instruction manual, or if a non-authorized service tries to open and / or repair the device.

4. IMPORTANT SAFEGUARDS

 **Read all instructions carefully before use!**

4.1 PREPARATION

- Use the equipment only for its intended use as described in this manual.
- Use the equipment with medications only under the instruction of your physician
- The device, including accessories, should not be shared by two different persons.
- Do not use the equipment if it has any damaged parts, or it has fallen into water. Return the product to a service center for examination and repair.
- Do not use extension cords.
- Keep the device, accessories and power cord away from heat sources and hot surfaces.
- Do not insert any objects into the slits or vents. Do not block the air flow.
- Always unplug this product before filling and after use. To use the equipment for the first time, or after storing it for an extended period, be sure to clean and sterilize all necessary parts as described on page 7.
- Do not handle the power cord with wet hands or immerse the AC adapter. An electrical shock may result.
- **The medication cup and mask (or mouthpiece) should be checked before each use. Replace with a new one if they are dirty or worn out.**
- Do not fill medication cup over 12 ml or the nebulizing rate will be weak.
- **Close supervision is required when this equipment is used by children or invalids.**
- Stop using the equipment immediately if any abnormality occurs.
- Keep your eyes away from the output of medication mist.
- Do not use the equipment where flammable gas is being used.
- Never operate the equipment with empty water tank or empty medication cup.
- Do not use solutions other than water for the tank. It may damage the transducer.
- Do not tilt or shake when operation.
- Never open the water reservoir cover or nebulizing compartment cover during operation. It may cause a severe injury.
- Some medications may not have good nebulization. Check with the physician or manufacturer of medication.

4.2 OPERATION

- Never immerse the equipment in water. It may damage the unit.
- Clean all necessary parts after each use as described on page 7.

4.3 STORAGE

- Do not store the unit under direct sunlight, high temperature or humidity.
- Keep the unit out of reach of small children.
- Always keep the unit unplugged while not in use.

EN

5. OPERATION PROCEDURE

- To use the equipment for the first time, or after storing it for an extended period, be sure to clean and sterilize all necessary parts as described on page 7.
- Always unplug this product and turn off the switch at bottom before filling and after use.

5.1. FILL WATER RESERVOIR

Place the device on a stable and level surface.

- A. Take out the cover of water reservoir as Fig. 1 (Nebulizing compartment will be taken out as a whole as well), and don't add water over the water level indicator as Fig. 2.
- B. Fill water reservoir with tap water or distilled water.

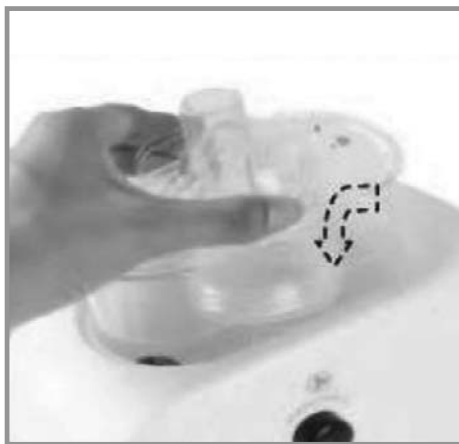
Fig. 1

Do not add water over water level indicator
(Coloured ring with a „WATER LEVEL“ sign)

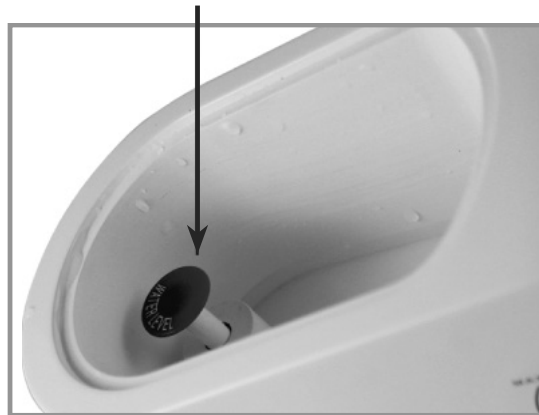
Fig. 2

Caution:

- Use of cool water will result low nebulizing rate.
- For higher nebulizing rate, use warm water (over 30 °C).

Do not use solutions other than water for the reservoir. It may damage the transducer.

5.2. FILL MEDICATION CUP

Screw off the cover of nebulizing compartment as Fig. 3 (If you want to clean or change the medicine cup, please screw off the bottom shell of nebulizing compartment as Fig. 5 and Fig. 6).

Fill medication directly into medication cup as Fig. 4.
Replace the cover of nebulizing compartment.

Fig. 3

Fig. 4

Fig 5

Fig. 6

CAUTION:

**Do not fill more than 12 ml, otherwise the nebulizing rate will be affected.
Use medications only under instructions of your physician.**

5.3. CONNECT EXTERNAL POWER

Plug the power cord into an AC outlet.

5.4. TURN ON THE POWER AND ADJUST NEBULIZING RATE

- A. Make sure water reservoir and medication cup are properly filled, and the unit is placed on a flat and dry surface.
- B. Turn the O/I Knob from 'off' position to the range between MIN and MAX, and the power indicator should be lit. Screw to the reasonable position to get your favorite nebulizing rate.

Please adjust the air flow rate by turning the knob (0-1-2) placed on the side of the device.

5.5 ADJUST THE AIR FLOW RATE

Turn the 'Air Flow Adjusting Knob' in the range between MIN and MAX to get your favorite air flow rate.

EN

Fig. 7

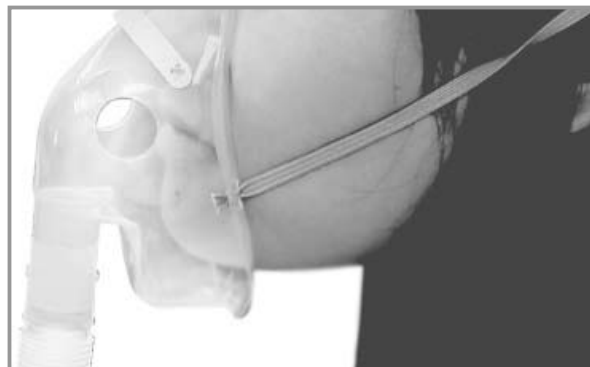
Fig. 8

CAUTION:

- When little water is left in the water reservoir, the equipment will turn off automatically and water level indicator will be lit to remind the user to add water.

5.6 FINISH OPERATION

Turn off the equipment by switching the O/I button to the 'off' position.

6. DISPOSE EXCESS MEDICATION AND WATER

- A. Take out the cover of water reservoir together with the nebulizing compartment.
- B. Remove the cover of the nebulizing compartment.
- C. Pour remaining medication out of medicine cup.
- D. Drain water from the water reservoir.

**CAUTION:**

Do not touch water, it is hot after use.

EN

7. CLEANING

Mask, mouthpiece, extension tube, water reservoir cover, nebulizing compartment, medication cup: Clean with mild detergent or commercially available disinfectant. Rinse all parts thoroughly and let air dry.
Main body: Clean with a soft cloth moistened with water and non-abrasive cleanser.

Desinfecting of the device

We suggest desinfection of the device as follows:

Wash all used accessories (medication cup, mouthpiece, nebulizing mask and inhalation hose) with appropriate desinfecting chemicals which are suitable for plastic materials accessible in pharmacies. Please follow the manufacturers instructions for dosage of the desinfecting agent. Carefully rinse everything with running water and dry.

8. SPECIFICATIONS

Product Category:	Aerosol Therapy Equipment
Product Description:	Ultrasonic Nebulizer
Model (kód):	Celimed U-615 (NB-152U)
Power Supply:	220–240 V AC 50/60 Hz
Power consumption:	cca 40 VA
Safety Classification	Type BF apparatus. Class II
Ultrasonic frequency:	1,7 MHz
Max. nebulizing rate:	≥ 2,0 ml/min
Particle size:	≈ 5 μm
Medication cup capacity:	max. 12 ml/min
Noise Level:	≤ 55 dB
Operation temperature and humidity:	+ 10 °C to + 35 °C, Rh 75% and below
Storage temperature and humidity:	- 10 °C to + 40 °C, Rh 95% and below
Dimensions:	199 mm × 105 mm × 178 mm
Weight:	cca 1400 g (Without accessories)
Accessories:	Extension tube (2 pieces), Medicine cup (5 pieces), Child & Adult masks, Mouthpiece (2 pieces), sealing ring (1pieces).

EN

9. TROUBLESHOOTING

Problem	Cause and/or solution
Unit does not operate when O/I button is turned on	Check that power cord is plugged in firmly.
	Check that enough water is filled to the water reservoir.
No nebulization or low nebulization rate	Check that there's medication in the medicine cup.
	Check that the medication cup is not broken or transfigured.
	Check medication level in medication cup.
	Check that transducer is clean.

<p>Manufacturer:</p> 	<p>HONSUN (NANTONG) CO., LTD. No.8, Tongxing Road, Economic&Technical Development Area, 226 009 Nantong City, Jiangsu, P. R. China</p>
<p>EU-representative:</p> 	<p>Shanghai International Holding Corp. GmbH (EUROPE) Eiffestrasse 80, 20537 Hamburg, GERMANY</p>
<p>Importer for the Czech Republic and authorized service:</p> 	<p>Celimed s.r.o. Sociální péče 3487/5a, 400 11 Ústí nad Labem service and repairs after warranty – tel.: 475 208 180 e-mail: info@celimed.cz www.celimed.cz</p>